Tubes Radiogènes à Anode Tournante  
Drehanoden - Röntgenröhre  
Tubos de Rayos - X con Ánodo Giratorio

Note: Document originally drafted in the English language.

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
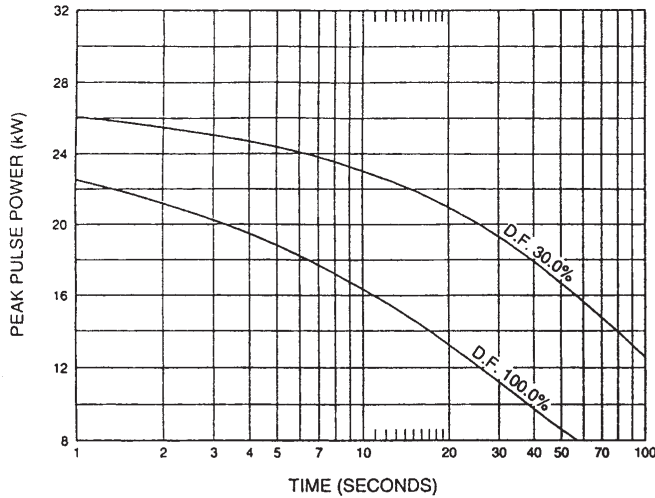
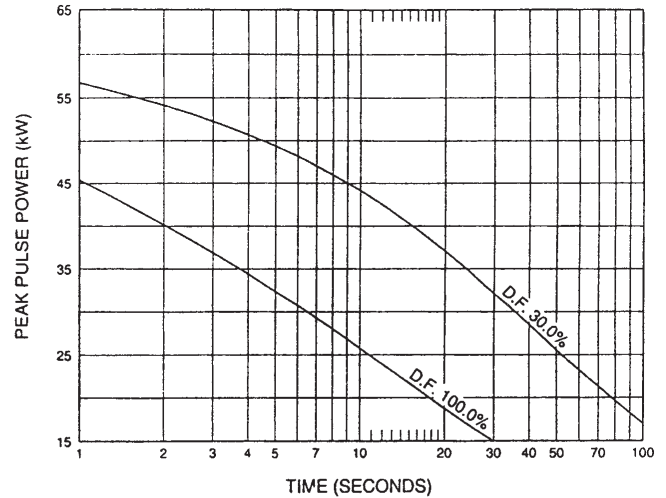
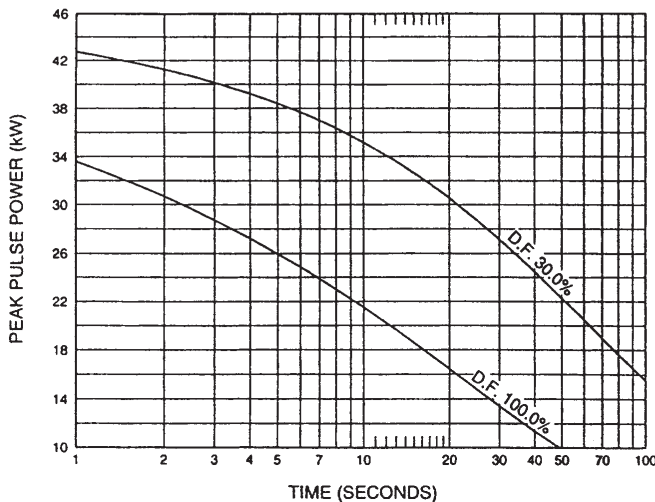
**Nominal Focal Spot Size - 0.8** 

**Nominal Focal Spot Size - 1.5** 

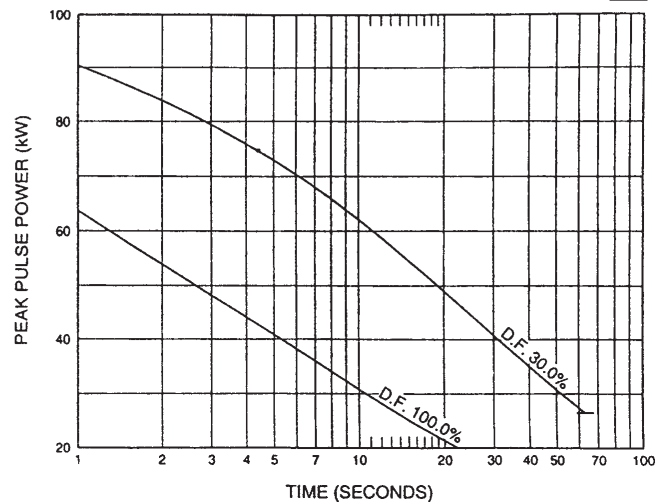**Note:**  
Rating charts reflect maximum tube performance. Tube operation is ultimately limited by system software.

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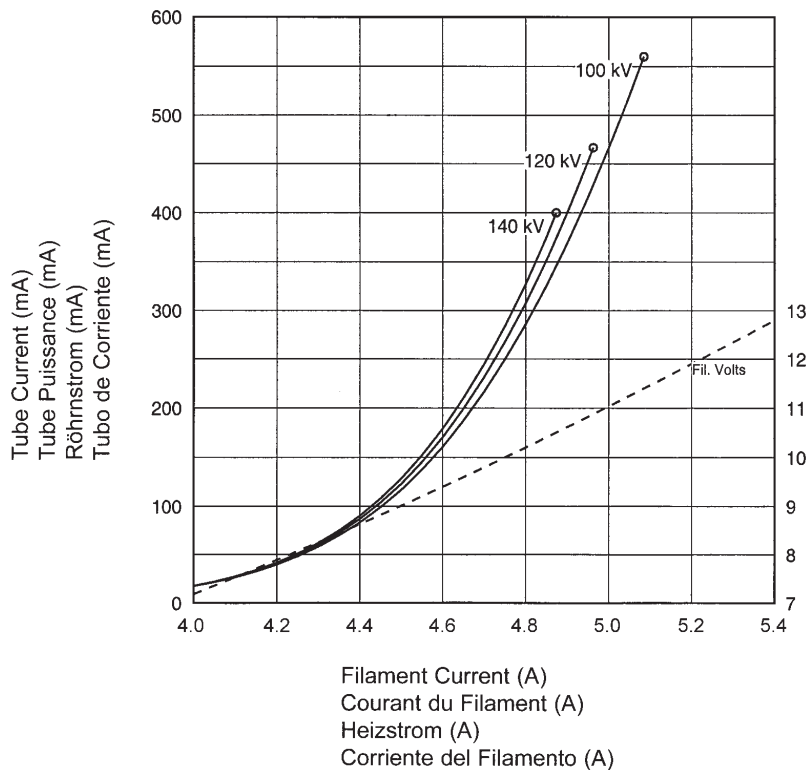
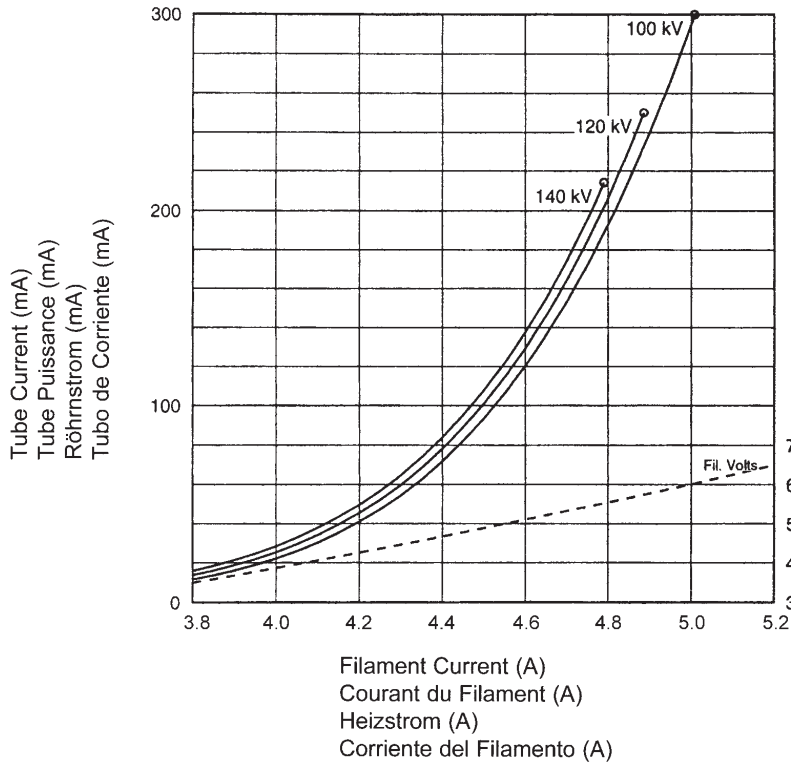
**Le Gaine B-160H**

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**Encaje de B-160H**

Maximum Peak Voltage .....	150 kV
Anode to Ground .....	75 kV
Cathode to Ground .....	75 kV
Maximum X-ray Tube Assembly Heat Content .....	1.1 MJ (1.5 MHU)
Maximum Continuous Heat Dissipation (Includes stator heat) .....	1,490 W (2,080 HU/sec)
Maximum Heat Exchanger Dissipation .....	1,800 W (2,540 HU/sec)
Focal Point Position (Central Ray) Within 1mm (X,Y Direction from the center of radiation port.)	
X-Ray Tube Assembly Permanent filtration .....	1.0 mm Al IEC 60522
Loading Factors for Leakage Radiation .....	150 kV, 4 mA
Federal Standard High Voltage Cable .....	72
Ambient Air Temperature Limits for Operation .....	5°C to 40°C
Temperature Limits for Storage and Transport .....	-20°C to +75°C
Humidity .....	+10% to +90%
Atmospheric Pressure Range .....	70 kPa to 106 kPa
Weight - Housing .....	23 kg (50.7 lbs)
Heat Exchanger .....	11.5 kg (25.3 lbs)
IEC Classification .....	Class 1
Safety Devices: Thermal Switch Normally Closed Contact .....	Opening at 85°C
Flow Switch - Normally Open contact Contacts close with adequate oil flow.	
Filament Frequency Limits .....	50 HZ - 400 HZ
Power Supply .....	DC

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
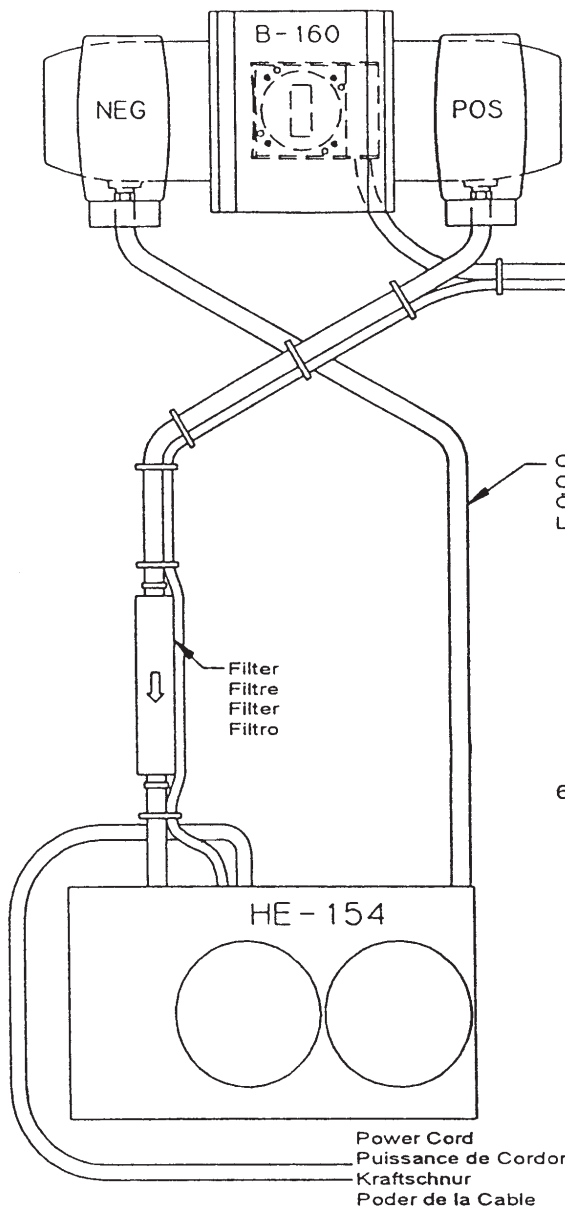
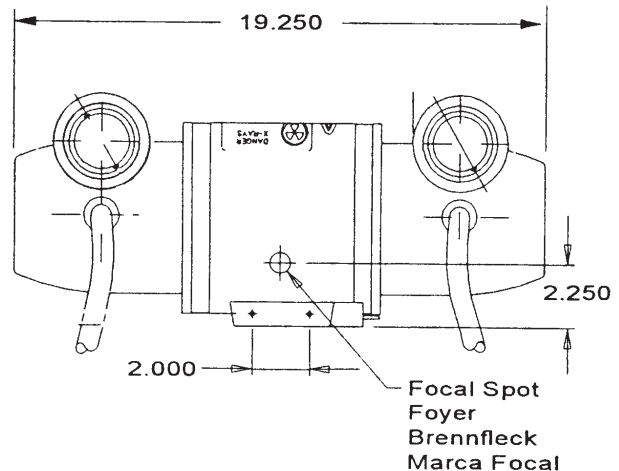
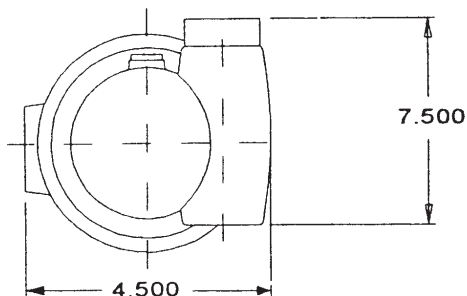
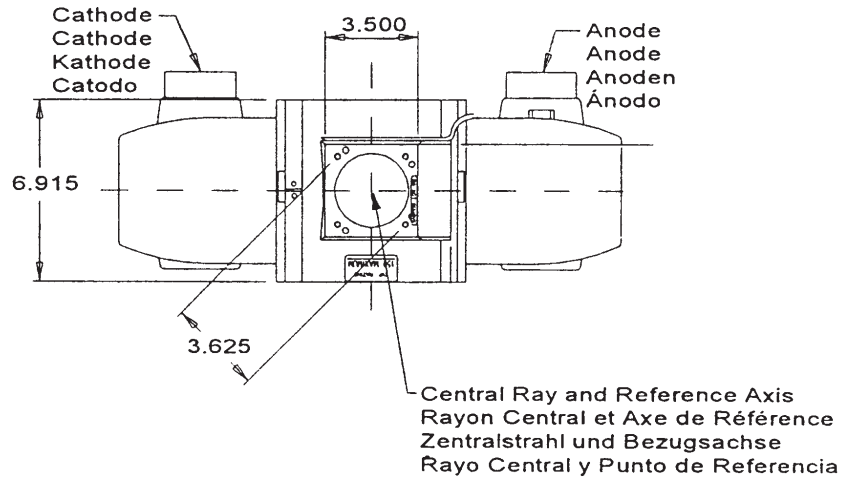
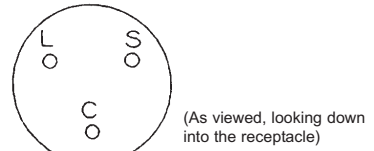
## TUBE HOUSING ASSEMBLY HEATING AND COOLING

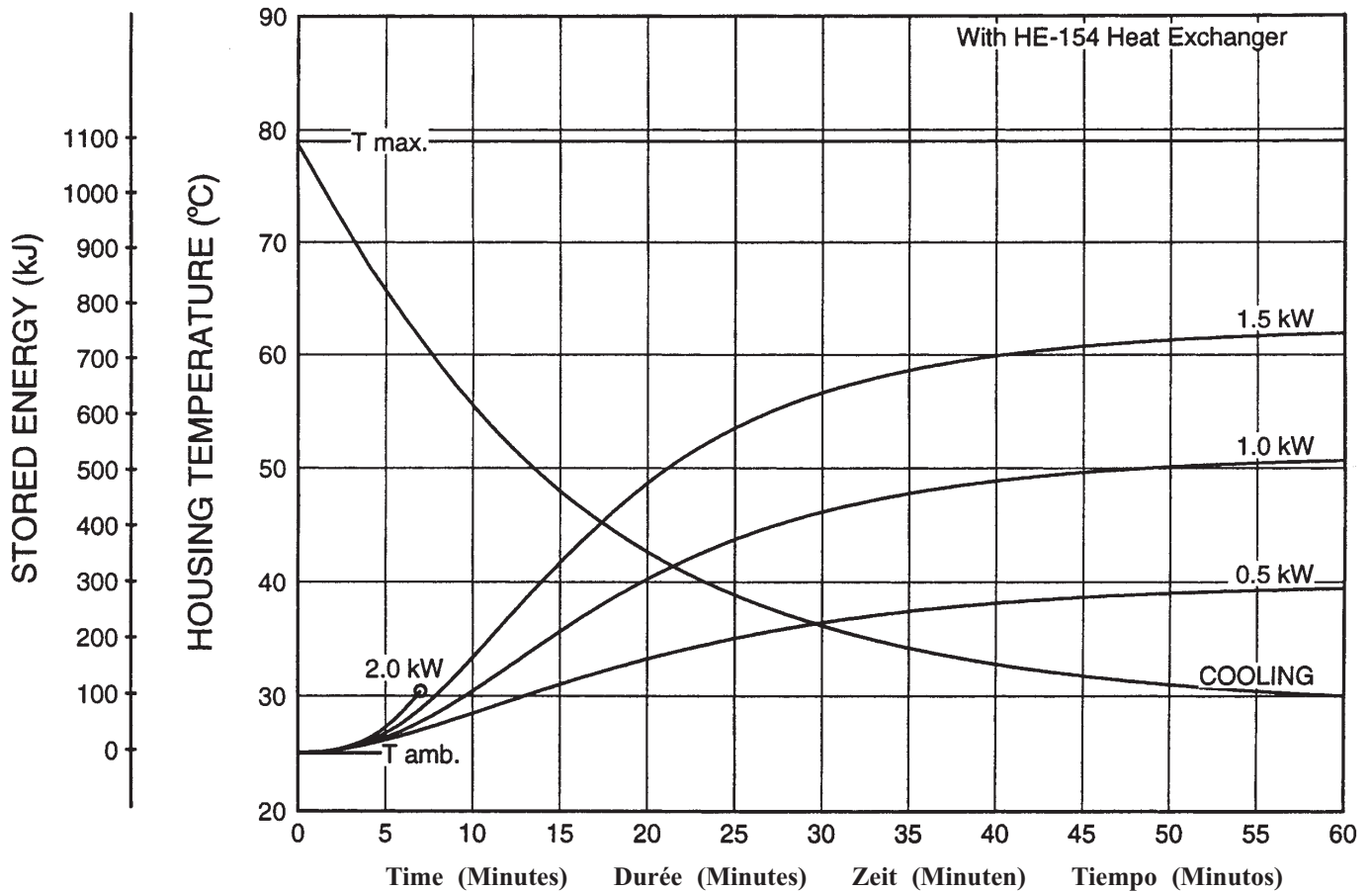**Note:**

- Heat inputs into housing include tube power, filament power, and stator power.
- Heating curves based on no restrictions of natural convection around tube housing assembly.
- Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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
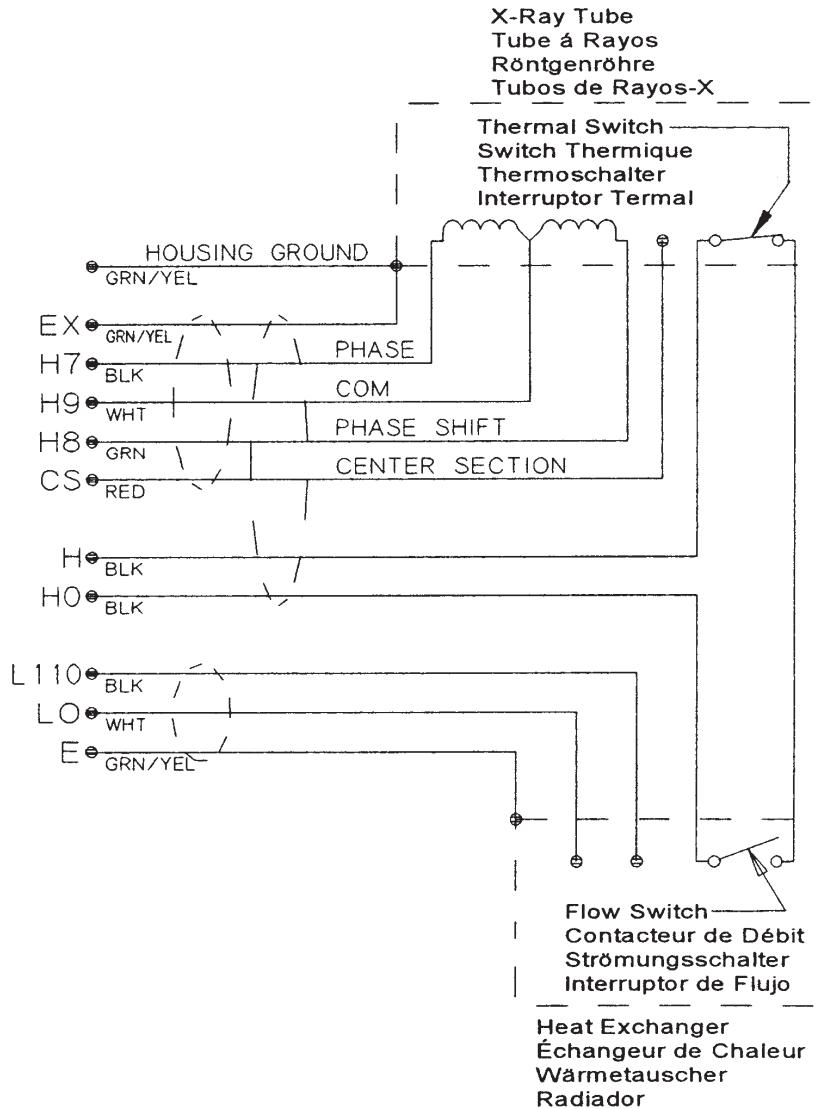
<b>Stator Type:</b> "R" Stator		
<b>Stator Coil Resistance:</b>		
Black to White	16.0 Ohms ±15%	
Green to White	50.0 Ohms ±15%	
<b>Starter Voltage:</b>		
50/60 Hz	<u>Start</u> 265 VAC	<u>Run</u> 75 VAC
150/180 Hz	600 VAC	115 VAC
<b>Time to Full Speed:</b>		
50/60 Hz	0 - 2700 RPM	6.3 Sec.
150/180 Hz	0 - 8000 RPM	6.3 Sec.
<b>X-Ray Tube Assembly:</b>		
S-780/B-160H	IEC 60601-2-28	

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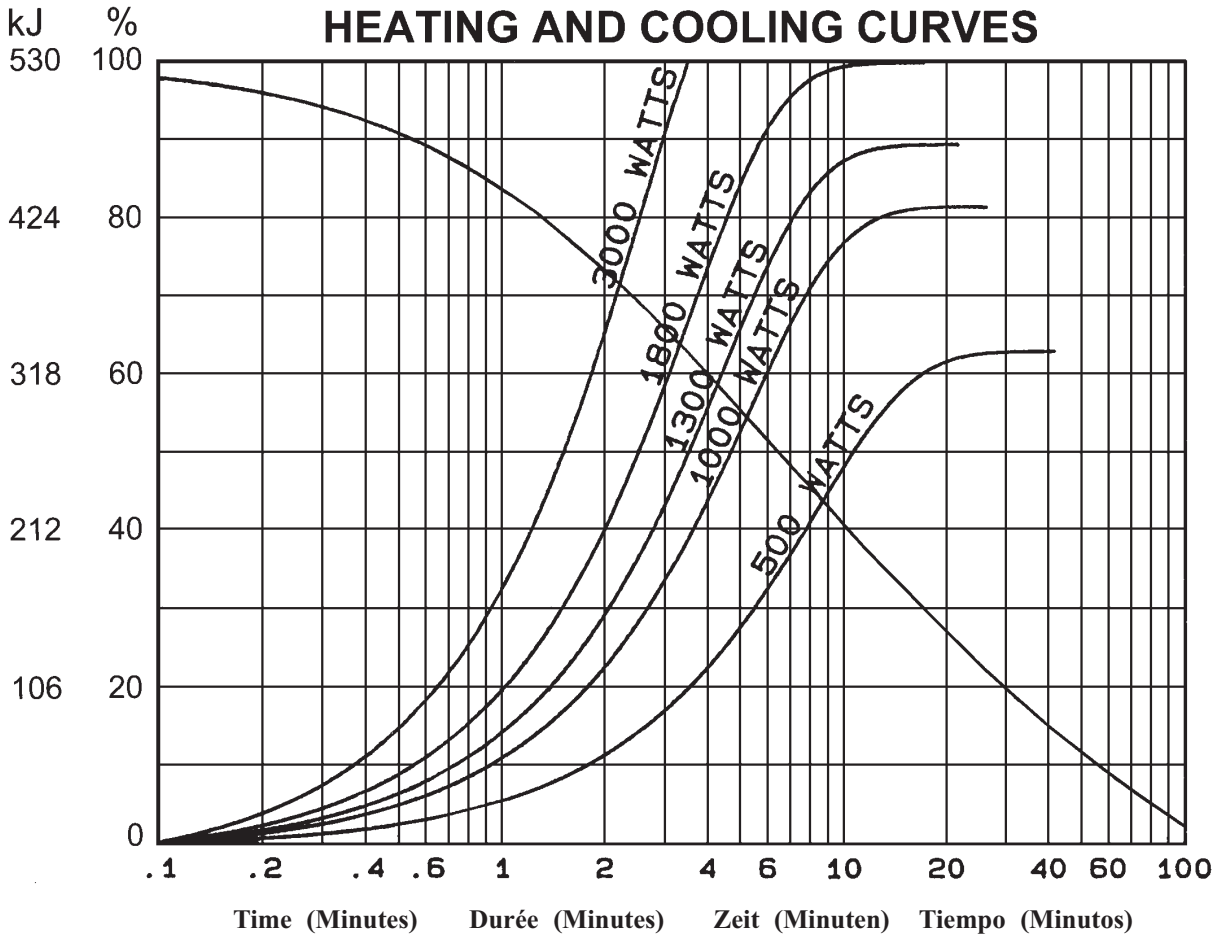**Note:**

Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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